

Viscot Medical Surgical Markers

Directions for use:

- Use an alcohol swab to remove the natural oils of the skin prior to making the mark on the skin.
- Allow area to dry.
- Mark the anatomy as required, and allow to dry.
- Prep patient with antiseptic as required for surgery.
- Post-surgery, remove ink with vigorous scrubbing using an alcohol swab.

Precautions:

Skin surface should be clean and dry prior to making the mark.

Use the mark as a landmark and do not incise over.

The marker is indicated for external use.

The marker is intended for single patient use. Do not re-sterilize.

Further Information can be found at www.viscot.com