

**soffzone™ Silicone Safety Products**  
**NEUTRAL ZONE SYRINGE HOLDER**  
**FSY309RED**

**GUIDELINES FOR DISINFECTION AND STERILIZATION**  
**(Minimum Requirements)**

**CLEANING AND DISINFECTION**

1. Clean with enzymatic cleaner as necessary to remove gross contamination. Clean drain holes with disposable brush if needed. Rinse with water.
2. Disinfect in Washer/Disinfector using standard temperatures and time cycles as recommended by manufacturer.

**STERILIZATION**

1. Wrap device separately for sterilization.
2. Discard device if damaged to such an extent that cleaning and sterilization would be compromised.

STERILIZER	Item	Exposure Time @250F (121C)	Exposure Time @270F (132C)	Drying Times (minimum)
<b>AUTOCLAVE Gravity Displacement</b>	wrapped	30 min	-	45 min  <b>OR</b>  30 min dry time + 15 min cracked door time + 30 min cool down time
<b>AUTOCLAVE Dynamic-Air- Removal</b> <small>(e.g., pre-vacuum)</small>	wrapped	-	4 min	45 min  <b>OR</b>  30 min dry time + 15 min cracked door time + 30 min cool down time

Viscot Medical PO Box 351, 32 West St. East Hanover, NJ 07936  
 email info13@viscot.com phone 800.221.0658 fax 973.887.3961 web viscot.com

rev 3 12 Jan 15 SBL

**soffzone™ Silicone Safety Products**  
**NEUTRAL ZONE SYRINGE HOLDER**  
**FSY309RED**

**INSTRUCTIONS FOR USE**

The *soffzone™* Neutral Zone Syringe Holder is designed to act as a clearly defined neutral zone area for the placement of syringes and other sharps instrumentation during surgery.

1. The Circulating Nurse will open the outer sterilization wrap and present the NEUTRAL ZONE SYRINGE HOLDER to the Scrub Technician.
2. The Scrub Technician will receive the sterile NEUTRAL ZONE SYRINGE HOLDER and place it either on the mayo stand or the back table (depending upon procedure being performed).

**NOTES:**

The NEUTRAL ZONE SYRINGE HOLDER is not designed for placement directly on the operative field/patient. It should also not be used as a passing device.

The silicone NEUTRAL ZONE SYRINGE HOLDER can be cleaned and re-sterilized via autoclave for multiple uses. Refer to Cleaning and Sterilization Guidelines.

Discard NEUTRAL ZONE SYRINGE HOLDER if damaged to such an extent that cleaning and sterilization would be compromised.

Viscot Medical PO Box 351, 32 West St. East Hanover, NJ 07936  
 email info13@viscot.com phone 800.221.0658 fax 973.887.3961 web viscot.com

rev 3 12 Jan 15 SBL