



Silicone Safety Products

NEUTRAL ZONE SYRINGE HOLDER
FSY309RED

**GUIDELINES FOR DISINFECTION AND STERILIZATION
(Minimum Requirements)**

CLEANING AND DISINFECTION

1. Clean with enzymatic cleaner as necessary to remove gross contamination. Clean drain holes with disposable brush if needed. Rinse with water.
2. Disinfect in Washer/Disinfector using standard temperatures and time cycles as recommended by manufacturer.

STERILIZATION

1. Wrap device separately for sterilization.
2. Discard device if damaged to such an extent that cleaning and sterilization would be compromised.

STERILIZER	Item	Exposure Time @250F (121C)	Exposure Time @270F (132C)	Drying Times (minimum)
AUTOCLAVE Gravity Displacement	wrapped	30 min	-	45 min OR 30 min dry time + 15 min cracked door time + 30 min cool down time
AUTOCLAVE Dynamic-Air-Removal (e.g., pre-vacuum)	wrapped	-	4 min	45 min OR 30 min dry time + 15 min cracked door time + 30 min cool down time

OP-marks, Inc. PO Box 374,35A West St. East Hanover, NJ 07936
phone 866.307.2757 fax 866.307.2758 web opmarks.com

Viscot Medical PO Box 351, 32 West St. East Hanover, NJ 07936
email info13@viscot.com phone 800.221.0658 fax 973.887.3961 web viscot.com

soffzone[®]

Silicone Safety Products

**NEUTRAL ZONE SYRINGE HOLDER
FSY309RED**

INSTRUCTIONS FOR USE

The *soffzone*[®] Neutral Zone Syringe Holder is designed to act as a clearly defined neutral zone area for the placement of syringes and other sharps instrumentation during surgery.

1. The Circulating Nurse will open the outer sterilization wrap and present the NEUTRAL ZONE SYRINGE HOLDER to the Scrub Technician.
2. The Scrub Technician will receive the sterile NEUTRAL ZONE SYRINGE HOLDER and place it either on the mayo stand or the back table (depending upon procedure being performed).

NOTES:

The NEUTRAL ZONE SYRINGE HOLDER is not designed for placement directly on the operative field/patient. It should also not be used as a passing device.

The silicone NEUTRAL ZONE SYRINGE HOLDER can be cleaned and re-sterilized via autoclave for multiple uses. Refer to Cleaning and Sterilization Guidelines.

Discard NEUTRAL ZONE SYRINGE HOLDER if damaged to such an extent that cleaning and sterilization would be compromised.

OP-marks, Inc. PO Box 374,35A West St. East Hanover, NJ 07936
phone 866.307.2757 **fax** 866.307.2758 **web** opmarks.com

Viscot Medical PO Box 351, 32 West St. East Hanover, NJ 07936
email info13@viscot.com **phone** 800.221.0658 **fax** 973.887.3961 **web** viscot.com