

## APPLICATION:

1. Remove clear protective film from TAT Marker and apply face down on skin.
2. Moisten 2x2 gauze with tap water and apply to backing of marker for about 10 seconds (the patient can help hold it in place).
3. Slowly remove paper backing, leaving tattoo in place on the skin. Allow to air dry for another 10 seconds before touching.

OP-marks™  
medical

VISCOT  
MEDICAL, LLC.

## TAT Markers

Temporary tattoos for correct site marking that cannot be moved or altered once placed to ensure Joint Commission compliance.

Surgical Site TAT Markers  
Contents: 100 sheets  
Made in U.S.A.

Patient Focused. Eco Friendly.

## REMOVAL:

If desired the TAT Marker can easily be removed by applying Transpore Tape over the temporary tattoo and then simply lifting it off of the skin.

Alternatively, the TAT Marker can be removed by applying an alcohol pad to the temporary tattoo for about 5 seconds, and then wiping the marking off.

Tat Marker color additives are non-toxic and non-allergenic

## IMPORTANT:

To ensure proper transfer of tattoo skin may need to be cleaned with an alcohol pad prior to application

Do not make incisions through tattoo. Skin underneath is not considered to be adequately prepped.