

Viscot Medical XL Skin Marker

Instructions for Use

1. Remove oil from the patient's skin with alcohol or soap and water. Allow area to dry thoroughly.
2. Mark incision locations as necessary. Allow ink to dry thoroughly before prepping.
3. Prepare skin with Povidine-iodine (Betadine) solution to disinfect. Ink markings may fade slightly but remain legible.
4. Locate incision directly next to the marked line.
5. Remove markings with vigorous scrubbing using commonly used prep solutions (i.e. Betadine, Chloroprep, etc). An alcohol swab can be used as well. Markings will disappear naturally with bathing in a few days.

IMPORTANT

1. For single patient use only to avoid infection.
2. Dispose of after use.
3. Do not re-sterilize.
4. Skin surface must be clean and dried before marking skin.
5. Consider whether the patient is sensitive to Gentian Violet.
6. Protect the tip when using the skin marker and cap after use to avoid the ink volatilizing.
7. Please destroy if the product is expired.