Traditional Ink Surgical Marker

VISCOT®

MEDICAL, LLC.

Viscot Medical, LLC
32 West St
East Hanover, NJ 07936 • U.S.A
800.221.0658 • 973.887.9273 • FAX 973.887.3961
www.viscot.com • viscotcs@viscot.com

EC **REP**

P. J. Dahlhausen & CO GmbH, 50996
KÖLN, GERMANY

©2023 Viscot Medical, LLC
VLIT-45 Rev. 03

ENGLISH INSTRUCTION FOR USE

1. Use an alcohol swab to remove the natural oils of the skin; allow the area to dry
2. Mark the anatomy as required and allow markings to dry
3. Prep patient with antiseptic as required for surgery
4. Locate incision directly next to the marked line
5. Post-Surgery, remove ink with vigorous scrubbing using an alcohol swab

INTENDED PURPOSE: Marking anatomical sites on intact skin prior to surgical procedures.

INTENDED USER: To be used by clinicians on surgical patients.

PERFORMANCE CHARACTERISTICS: Clear visible purple mark.

CONTRAINDICATIONS: Rulers packaged with markers should be used as an indicator only and not for precise accurate measurements.

PRECAUTIONS

- Marker is for External Use, not internal use.
- Not for ocular use
- Not for use on newborns
- Reuse of single use device could result in infection/contamination and/or device failure which could lead to patient harm.

HANDLING: No special requirements needed

NOTICE:

Any serious incident that occurs with the device should be reported to the manufacturer and the Competent Authority of the Member State in which the user/patient is established.

Product contains Gentian Violet ink

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