

## X-Site<sup>™</sup>

Correct site skin adhesive for endourologic procedures.



X-Site<sup>™</sup> Radiopaque Ink on skin adhesive  
Patent Pending. Made in the U.S.A.

The leader in surgical marker innovation

### PRECAUTIONS:

1. Ensure the device is applied to the correct side(s)
2. Adhere tightly to the skin; if needed, shave body hair beforehand
3. Use care when removing device from patient, especially the elderly or those with weakened skin
4. Do not apply over open wounds or compromised skin
5. Do not reuse after removing from the skin

VLIT-42 Rev-2



### **APPLICATION:**

1. Remove X-Site from its liner by bending the corner to expose the adhesive
2. Apply X-Site on the patient's skin 1.5"- 2.0" to the left or right of the umbilicus, over the procedure site
3. Use a surgical marker to mark the site as per Joint Commission guidelines
4. X-Site will be visible on the fluroscopic image